

Scope “DEEP CLEAN” 2

Valisafe
clearly a better indicator



Easy to
use

Concentrated Enzyme & Contoured Sponge Kit

- For Pre “AER” Decontamination
- Pre Measured Wash Dose and Sponge
- Single use prevents cross contamination
- pH neutral enzyme detergent
- Contoured “scope shaped” sponge
- Makes 3 Gallons / 12 Litres in basin
- Non-toxic

NOTES – “best practice” guidance

- Modern AER machines use spray force and high flow rates to dislodge debris, these supplement manual cleaning but do not substitute it’.
- Manual cleaning should be performed in warm water with added enzymatic cleaner – therefore the removal of deposits by the AER is less critical.
- Wherever possible single use chemicals should be used as it avoids many problems associated with control of reuse.
- Choose a neutral enzyme to avoid scope damage.
- Use a super glide contoured sponge for fast efficient wipe down.
- Complete immersion of scope and flushing is required to achieve effective “pre clean” prior to AER.

Order Details

M20191

Scope “Deep Clean”

100ml 3E-Zyme - Concentrated
1x Super Glide scope sponge